37426/1724

VXSE® ONE

For dental professional use only.
Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

INSTRUCTIONS FOR USE / Gebrauchsinformation / Mode d'emploi / Gebruiksaanwijzing / Инструкция по использованию / Istruzioni per l'uso / Instrucciones de uso / Kullonnik Käyttösuu / Instruções de uso / Instrukcja użyzania / Инструкция із застосування /取扱説明書 / تعليمات الاستخدام

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ENGLISH

Product Description
VXSE® One is an elastomeric-precision impression material with vinyl polysiloxane and polyether materials (Vinylsiloxane-ether). VXSE® One has medium viscosity and is available in 380-ml cartridges. 5:1, for use in the usual automatic dosing and mixing systems, e.g. Sympress.

VXSE® One is to be used as a medium bodied tray or syringeable impression material in one-step technique (monophase) for:

- Imprint impressions
- Fixation impressions
- Functional impressions
- Impressions for crowns/bridges, inlays/onlays and veneer preparations

Patient Target Group
Patients being treated in the context of a dental procedure.

Warnings
Do not use VXSE® One for two-step impressions (two-stage impression technique) or temporary relinings. Do not mix or use VXSE® One with other elastomeric impression materials.

Potential Allergic Substances
Platinum compound, 4-methoxyphenol.

Cautions
Do not use after expiration date.

Normally, impression materials do not cause allergic reactions; however, an allergic reaction cannot be ruled out in sensitive individuals.

If in doubt, we recommend performing an allergy test before use.

Do not use the material in case of known allergies to any of the ingredients or in case of contact allergic reactions. Do not leave any residual impression material in the sulcus or oral cavity.

Do not swallow impression material! If swallowed, consult a medical doctor. If problems arise or persist.

Always contact with the eyes. If there is accidental contact with the eyes: Rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

Use of products containing active sulfur, aluminium chloride or nitrogen compounds (retraction corks containing reticulated polyethylene impression material, etc.) in conjunction with this product will interfere with the setting reaction of the Vinylsiloxane-ether® material. Use of these materials requires the area to be rinsed thoroughly to remove all residue.

Always contact with latex gloves. After each change of apparatus, it is to be rinsed with water.

When taking impressions of areas with severe undercuts and wide interdental spaces, standard blocking-out techniques must be used.

When using a custom impression tray, ensure that sufficient space remains between the side of the tray and the teeth/aw. Do not allow the material to enter the water or styrene system to avoid environmental hazard.

Always contact with clothing, as the material cannot be removed by dry cleaning.

Impression tray: Preparation and Advheses

In general, all conventional impression trays can be used if the necessary pressure build-up can be generated.

For improved retention (adhesion) to the individual tray and the impression material, the individual tray can be sand-blasted or mechanically roughened prior to brushing on Kettbach Universal Adhesive.

Allow the adhesive to dry in accordance with the information in the instructions for use and cover with sufficient impression material in the impression tray.

Instructions for the Safe Handling of the 380-ml cartridge system; 5:1

Use the tray to use please unscrew the plug counter-clockwise and pull it out. Check the outlet opening. Then screw on the mixer.

Please use only the dispensing and mixing components compatible with the 380 ml cartridge system for dispensing and mixing impression material.

The 380 ml cartridge is heavy and could be damaged if dropped. Damaged cartridges must not be used under any circumstances.

When using the material in a Sympress I dispensing device, please activate the speed setting indicated on the product label. Failure to comply with this instruction may modify the curing characteristics of the impression materials.

If you are using a unit other than the Sympress mixing device, please follow the instructions for the automatic dispensing and mixing system you are using.

When changing to a different mixing device, always discard at least 3 cm of mixed material.

Using the 380-ml cartridge system; 5:1 in Dispensing Device Sympress I / Sympress II

By pressing the plunger return knob, allow the plunger to return to its original position automatically (while the cover is closed). Next, open the cover.

Remove the tray from the pre-assembled mixer in the Sympress I/Sympress II dispensing and mixing unit. If you are working with a previously used cartridge, the filled mixer will already be located on the cartridge as a cap.

If you are using a Sympress I dispensing device, please activate the speed setting indicated on the product label. A light signal will confirm your selection.

Close the device cover.

Using the Cartridge for the First Time

Dispense the impression material (ensure that the mixer is wound into place fully) until the base material and catalyst are mixed homogeneously, discard at least the first 3 cm when dispensing.

At least 3 cm of the mixed material must be discarded each time the plunger is restarted or when using the cartridge in different mixing units.

Load the impression tray (or application syringe) with the required amount of material by pressing the appropriate feed key. Hold the impression tray at an angle when loading and press against it lightly. Leave the mixer in the material when dispensing to avoid air bubbles.

Release the tray fast as soon as the required amount of material has been dispensed. The filled mixer should remain on the cartridge as a seal.

Remove the used mixer before the next application by turning it to the left, visually check the outlet opening for blockage and attach a new mixer.

Further Use of a Cartridge Already in Use

Remove the tray from the cartridge and begin the new application (material no longer has to be discarded). Proceed with the application as described above.

Storage Conditions

Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations.

Disinfection

Impressions can be disinfected using for example a 2% acidic glutaraldehyde solution. Use disinfection solutions specific for impression materials. See manufacturer instructions.

Model Preparation

Impressions with VXSE® One may be poured immediately after disinfection of the impression up to several weeks with high quality dental stone (class III – V). Impressions can also be poured several times.

Composition

Vinyl polysiloxanes, Hydrogen polysiloxanes, Filler, Pigments, Additives, Platinum compound.

Disposal

Completely emptied cartridges / Mixers / applicator tips and set material can be disposed in the household rubbish.

Trademarks

VXSE® and Vinylsiloxane-ether® are registered trademarks of Kettbach GmbH & Co. KG.

Note

Selected Kettbach materials are available in certain markets only.

For dental professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

Limitation of Liability

Except where prohibited by law, Kettbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

Updated: April 16, 2024

Product Overview

Product name	ISO 4823	Consistency (approx. mm)	Mixing ratio and total content volume per unit	Mixing technique	Mixing element
VXSE® One	Type 2 Medium bodied	38	5:1, 380 mL cartridge	Sympress/Sympress II or other automatic dispensing and mixing unit	Mixer

Product name	Working time at 23°C/74°F ≤	Working time at 35°C/95°F	Intracoral setting time at 35°C/95°F ≥	Total setting time* ≥
VXSE® One	2:30 min.	1:20 min.	2:45 min.	5:15 min.

* Total setting time (removal time from mouth) from start of mix

Note: To ensure optimal impressions, the temperature of the material should not deviate from 23°C (74°F) before applying. Otherwise, working and setting times will be affected.

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