Mucopren®

Mucopren® Soft

Mucopren® Silicone sealant

Mucopren® Adhesive

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Brush a thin coating of Mucopren® Adhesive onto the prepared and roughened surface and allow to air-dry for at least 90 seconds. Ensure that the adhesive is applied **beyond the prepared periphery**.
Apply Mucopren® Soft evenly using the dispensing gun (to prevent air being entrapped, keep the opening of the mixing nozzle open and the dispensing gun tilted to the denture base. The active and passive functional excursions can then be carried out. Processing: After removing from the patient's mouth, leave the denture and the relining at room temperature for three minutes before continuing to process. A water bath is no longer needed.

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ENGLISH

Product Description

Mucopren® Soft is a permanently soft durable relining material for removable dentures based on vinyl polysiloxane.

Indications / Techniques

Mucopren® Soft can be applied directly or indirectly

- for permanently soft linings
- for compensating for areas of mucous membrane with varying degrees of resiliency
- for relieving sharp bony ridges
- for relieving the operation site after placing implants
- for relining temporary restorations
- for conditioning the mucous membranes in cases with denture sores
- for fabricating and relining obturators

Patient Target Group

Patients being treated in the context of a dental procedure.

Warnings

Do not use Mucopren® Soft relining material as an impression material.

Do not use Mucopren® Soft products with other liners, sealants or adhesives.

Potential Allergenic Substances

Platinum compound, 4-methoxyphenol.

Cautions

Do not use after expiration date.

Normally, impression materials should not cause allergic reactions; however, an allergic reaction cannot be ruled out in sen-sitive individuals.

If in doubt, we recommend performing an allergy test before use.

Do not use the material in case of known allergies to any of the ingredients or in case of contact allergies.
Also, if used on patients whose immune system is impaired, or on those who are suffering from a variety of other dis-eases (such as diabetes mellitus) or those who are taking medications including antibiotics or cortisone, it has been ob-served in rare cases that micro organisms (e.g. candida albicans) may colonize the soft lining. In such cases, whenever the lining of the denture is replaced, soft lining should not be used since the recolonization by micro organisms is lik-ely if the patient status remains unchanged.

Increased colonization by microorganisms and the development of denture stomatitis have been observed in patients due to inadequate oral hygiene and improper cleaning of soft-lined dentures.
Please instruct the patient about adequate oral hygiene and provide him with the care instructions from the base set.
Do not leave remains of soft liner in the mouth.

Do not swallow this material if swallowed; consult a medical doctor if problems arise or persist.

Avoid contact with eyes. If accidental contact with the eyegoceros, rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

Solutions used prior to the application of Mucopren® Soft (e.g., retraction solutions, mouth rinsing solutions) may interfere with the setting reaction of the relining material and must be removed by thorough spraying and drying.
If gloves are worn when processing the material, they should first be tested for compatibility using a sample. Some types of gloves can disturb the activity of the catalyst.

Should additional soft liner require additional, relieve the area liberally and apply another coat of Mucopren® Soft. Brush one coat of adhesive onto the exposed areas of acrylic only.

When handling chemicals the usual precautions must be taken.

Do not allow the material to enter the sewer or water system to avoid environmental contamination.

Avoid contact with clothing, since the material cannot be removed by dry cleaning.

Note

To ensure optimal relinings, the temperature of the material should not deviate from 23°C/74°F before applying. Otherwise, working and setting times will be affected.

Please Note – Mucopren® Adhesive

Once the bottle of Mucopren® Adhesive is immediately after use.

Push the plunger into the cartridge and discard a small quantity of the relining material. Using the dispensing gun, immediately apply Mucopren® Soft evenly onto the denture base.

Replace the plug after use. Before using the cartridge again, remove the cartridge cap or the filling mixing tip and check the cartridge outlet openings to ensure that no polymerized materials have sealed the openings. Carry cloggyed openings and dispense a small amount of relining material until equal amounts are extruded at the same rate.

Install a new mixing tip and continue as described.

Cleaning and Disinfection of the Applifyx 4

Remove the cartridge cap.

Remove any uncured paste using a cloth soaked in alcohol.

The Applifyx 4 dispensing gun can be used several times if properly disinfected. Kettenbach recommends exchanging the gun if there are visible trace of use. Disinfection can be carried out using a commercially available dipping disinfectant. Kettenbach recommends the use of glutaraldehyde-based solutions.

Instructions for Use for the Direct and Indirect Methods

Mucopren® Soft was developed specifically for the use of cold and hot polymerizing denture base plastics. Wash the denture base thoroughly with water and dry it. Use it to reduce the surface to allow for the thickness of the soft liner required. To create a neat vestibular junction between the soft liner and denture acrylic, it is advisable to trim a right-angled periphery. Brush a thin coating of Mucopren® Adhesive onto the prepared and roughened surface and allow to air-dry for at least 90 seconds. Ensure that the adhesive is applied **beyond the prepared periphery**.

Apply Mucopren® Soft evenly using the dispensing gun (to prevent air being entrapped, keep the opening of the mixing nozzle open and the dispensing gun tilted to the denture base. The active and passive functional excursions can then be carried out. Processing: After removing from the patient's mouth, leave the denture and the relining at room temperature for three minutes before continuing to process. A water bath is no longer needed.

Dentures which are not made of standard self-curing acrylic should be immersed in lukewarm water (80°C/186°F–35°C/95°F) for 2 hours.

Once the material has cured, cut the excess material off with scissors, a scalpel or rotary instrument to create a neat junction between the denture acrylic and softliner. Cross-cut steel burrs for soft linings (e.g. Figur 82060 from Bredt) are useful for bulk reduction.

Finishing can be carried out with the polishing discs (supplied with the basic set. Trimmed areas of acrylic can be polished as usual. Do not use polishing paste that leaves residual grease. Always clean the surfaces with hot water (steam cleaner) and then before sealing.

Lastly, apply a thin layer of Mucopren® silicone sealant to the prepared area of soft liner. It is advisable to place a small amount of Mucopren® silicone sealant into a suitable container before use. Place the reling jig or flask in a pressure vessel or a water bath containing water at ≥ 50°C/122°F warmen Wasser geben.

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Disinfection

Soft lined dentures can be disinfected (immersion).
The cross-cut steel bur for permanent soft relining material (Figur 82060 from Bredt) that is included in the basic set is manufactured from normal steel and is therefore not suitable for reprocessing in a steam steriliser.

It is also important for normal steel to ensure that disinfectants and cleaning agents are used with corrosion protection.

Composition

Vinyl polysiloxanes, Hydrogen polysiloxanes, Filler, Pigments, Additives, Platinum compound.

Disposal

The empty cartridges/mixing tips/appliator tips and set material can be disposed in the household rubbish.

Trademarks

Mucopren® is a registered trademark of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials/products are available in certain markets only.

For dental professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

Limitation of Liability

Under no circumstances will be liable, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including war-ranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

Updated: 15 July 2025

Product Overview

Product name	ISO 10139-2	Mixing ratio and total content name per unit	Mixing technique	Mixing element
Mucopren® Soft	Type A	1:1, 50 ml cartridge	Applifyx 4 dispensing gun DS-50 1:1/2:1	mixing tip
Mucopren® silicone sealant	—	1:1, 50 ml cartridge	Applifyx 4 dispensing gun DS-50 1:1/2:1	mixing tip
Mucopren® Adhesive	not applicable	not applicable	—	brush

Technical Data

Product name	Mixing ratio	Working time at 23°C/74°F	Internal setting time at 35°C/95°F	Total setting time at 35°C/95°F	Hardness (ca.) Shore	Linear dimensional change (approx.) %	Elastic-Recy-tivity test (approx.) %	Strain in compres-sion (approx.) %
Mucopren® Soft	Type A	1:1, 50 ml cartridge	DS-50 1:1/2:1	DS-50 1:1/2:1	DS-50 1:1/2:1	DS-50 1:1/2:1	DS-50 1:1/2:1	DS-50 1:1/2:1
Mucopren® silicone sealment	—	1:1, 50 ml cartridge	DS-50 1:1/2:1	DS-50 1:1/2:1	DS-50 1:1/2:1	DS-50 1:1/2:1	DS-50 1:1/2:1	DS-50 1:1/2:1
Mucopren® Adhesive	not applicable	not applicable	—	—	—	—	—	—

Mucopren® Soft direkte Unterfüttung	1:1	2-15 Min.	3-15 Min.	5-30 Min.*	A 28	– 0,20	99,5	9,4
Mucopren® Soft indirekte Unterfüttung	1:1	2-15 Min.	not applicable	> 300 min. in the pressure vessel or water bath at ≥ 50°C/122°F	A 28	– 0,20	99,5	9,4
Mucopren® Versiegelungssilikon	1:1	1-30 Min.	not applicable	6-30 min. at 23°C/74°F	A 33	– 0,20	—	—

* Total setting time (removal time from mouth) from start of mix

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* Total setting time (removal time from mouth) from

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