Instructions For Use

Lastic[®]

Polysiloxane impression material ISO 4823

Made in Germany
37105/5019

Gebrauchsinformation
Instructions For Use
Mode d'emploi
Gebruiksaanwijzing
Инструкция по использованию
Istruzioni per l'uso
Instrucciones de uso
Kullanım Kılavuzu
Instruções de uso
Instrukcja użycia
Інструкція для застосування
取扱説明書
تعليمات الاستخدام

**Inhalt/Content/Contenu/Inhoud/содержимое/Contenuto/Contenido/
İçerik/Conteúdo/Treść/Зміст/목차/目次/الفهرس**

Deutsch	4	Polski	51
English	9	Українська	57
Français	14	日本語	63
Nederlands	19	68	عربي
Русский	24		
Italiano	30		
Español	35		
Türkçe	41		
Português	46		

Polysiloxane impression material DIN EN ISO 4823

Lastic® Xtra Putty

Lastic® function Medium

Lastic® 90 Fine

Lastic® Universal hardener paste

Manufacturer:

Kettenbach GmbH & Co. KG

Im Heerfeld 7

35713 Eschenburg, Germany

www.kettenbach.com

Produktbeschreibung

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Product description

Lastic® precision impression materials are condensation-curing elastomeric materials.

The Lastic® family includes three different viscosities (putty, medium bodied, light bodied), available in the standard 582 ml jar and in 155 ml tubes along with Lastic® Universal hardener paste (60 ml).

Indications for use

Lastic® Xtra Putty is used as a kneadable preliminary material for:

- Correction impression technique (two-step)
- Correction impression technique using a placeholder film
- Impression material for functional margin design

Lastic® function Medium is used as a medium-viscosity material for:

- Reline impressions
- Functional impressions

Lastic® 90 Fine is used as a low-viscosity preliminary material for:

- Functional impressions
- Reline impressions
- Correction impression technique

Warnings

Lastic® impression materials must not be used with addition-curing silicones, Vinylsiloxanether®, polyether, or polysulfide materials.

Lastic® Xtra Putty for precision impressions must always be combined with Lastic® 90 Fine.

Lastic® Xtra Putty impression materials are not suitable to be used alone for precision impressions.

Lastic® impression materials must always be combined with Lastic® Universal hardener paste.

Cautions

Do not use after expiration date.

Do not leave any residual impression material in the sulcus or oral cavity.

Do not swallow impression material! If swallowed: consult a medical doctor if problems arise or persist.

Avoid contact with the eyes. If there is accidental contact with the eyes: Rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

The product does not normally cause allergic reactions. However, for sensitive persons, an allergic reaction to the product cannot be ruled out.

Do not allow the material to enter the sewer or water system to avoid environmental hazard.

Avoid contact with clothing, since the material cannot be removed by dry cleaning.

Note

To obtain optimal impressions, the product temperature before use must not deviate significantly from 23 °C (74 °F). The processing and intraoral setting times may otherwise be affected.

Impression tray: Preparation and adhesives

In general, all conventional impression trays can be used if the necessary pressure build-up can be generated. If it is not possible to achieve sufficient retention with the impression material, brush the impression tray with Reto® adhesive. The surface of the impression tray must be repeatedly and fully

brushed so that the entire surface of the tray is covered. Allow the adhesive to dry in accordance with the information in the directions for use and cover with sufficient impression material in the impression tray.

Instructions for use:

A dose of the activator that deviates by up to 25% more or less shortens or prolongs respectively not only the setting time but also the processing time. Do not exceed the dose of the hardener paste by more than 25% under any circumstances.

Lastic® Xtra Putty (582 ml) for manual kneading

1. Brush the impression tray with a thin layer of adhesive (Reto® adhesive) for condensation-curing silicones. Allow to dry.
2. Remove the Lastic® Xtra Putty with the dispensing scoop from the jar and wipe off excess material.
3. Spread the Lastic® Xtra Putty on the mixing pad and make grooves in the putty using the kneading fork.
4. Dosing
 - With Lastic® Universal hardener paste (REF 16113)
 - For each dispensing scoop of impression material, add a 13 cm length of hardener paste (corresponding to the same length of the dispensing scoop).

5. Mixing

- Mix the Lastic® Universal hardener paste into the impression material within 30 seconds using the kneading fork and then knead for another 30 seconds until the color is homogeneous.

Lastic® function Medium (155 ml)

When using the tube for the first time, pierce through the membrane seal using the spike on the lid.

1. Dosing

- With Lastic® Universal hardener paste (REF 16113).
- Place strands of impression material and hardener paste of the same length onto the mixing pad.
- 10 cm strand length of impression material (7.9 ml) + 10 cm strand length of hardener paste (0.4 ml). Mixing technique: manually using the mixing spatula.

2. Mixing

- Mix the impression material and the hardener paste within 30 seconds using the mixing spatula until the color is homogeneous and free of bubbles.
- Mixing in air bubbles can be avoided by repeatedly spreading the material out flat while mixing.

Lastic® 90 Fine (155 ml)

When using the tube for the first time, pierce through the membrane seal using the spike on the lid.

1. Dosing

- With Lastic® Universal hardener paste (REF 16113).
- Place strands of impression material and hardener paste of the same length onto the mixing pad.
- 10 cm strand length of impression material (3.9 ml) + 10 cm strand length of hardener paste (0.4 ml). Mixing technique: manually using the mixing spatula.

2. Mixing

- Mix the impression material and the hardener paste within 30 seconds using the mixing spatula until the color is homogeneous and free of bubbles.
- Mixing in air bubbles can be avoided by repeatedly spreading the material out flat while mixing.

Lastic® Universal hardener paste is used as an activator in combination with all Lastic® impression materials. This combination must always be complied with to ensure the character and technical properties of the impression material.

Storage conditions

Store in a dry place, protected from sunlight and at room temperature, avoiding extreme temperatures.

Disinfection

Impressions can be disinfected with a 2% glutaraldehyde solution or with disinfection solutions specifically for impression materials. Follow the manufacturer's instructions.

Electroplating

The impressions may be silver-plated with an alkaline-plating solution only.

Model preparation

The impression can be poured within 2 to 24 hours using standard dental stone (classes III–V) to prepare models.

Trademarks

Lastic® and Reto® are registered trademarks of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials/products are available in certain markets only.

For professional use only.

Limitation of liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the legal basis asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

Updated: December 10, 2019

Product overview

Product name	ISO 4823	Consistency (approx.) mm	Content volume per unit	Mixing technique
Lastic® Xtra Putty	Type 0, Putty	24	582 ml in jars	Manual
Lastic® function Medium	Type 2, Medium bodied	34	155 ml from the tube	Manual
Lastic® 90 Fine	Type 3, Light bodied	44	155 ml from the tube	Manual
Lastic® Universal hardener paste			60 ml from the tube	Manual

Technical data

Product name	Working time at 23 °C / 74 °F ≤	Working time at 35 °C / 95 °F ≤	Intraoral setting time at 35 °C / 95 °F ≥	Total setting time* ≥	Hardness (approx.) Shore	Linear dimensional change (maximum) %	Elastic recovery test (approx.) %	Strain in compression (approx.) %
Lastic® Xtra Putty	1 minute	2 minutes	4 minutes	6 minutes	A 63	- 0.60	98.5	2.0
Lastic® function Medium	1 minute	2 minutes	5 minutes	7 minutes	A 33	- 0.60	99.0	8.0
Lastic® 90 Fine	1 minute	1 minute 30 seconds	5 minutes	6 minutes 30 seconds	A 27	- 0.70	99.3	9.0

*Total setting time (removal time from mouth) from start of mix.

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