	
Lastic® Universal hardener paste	<div><div><div></div><div>KETTENBACH</div></div></div> <div>Simply intelligent</div>

Hardener paste für Lastic® Xtra Putty, Lastic® function Medium und Lastic® 90 Fine.
Hardener paste for Lastic® Xtra Putty, Lastic® function Medium and Lastic® 90 Fine.
Durcisseur en pâte pour Lastic® Xtra Putty, Lastic® function Medium et Lastic® 90 Fine.
Uithardingspasta voor Lastic® Xtra Putty, Lastic® function Medium en Lastic® 90 Fine.
Паста-отвердитель для Lastic® Xtra Putty, Lastic® function Medium и Lastic® 90 Fine.
Catalizzatore in pasta per Lastic® Xtra Putty, Lastic® function Medium e Lastic® 90 Fine.
Pasta endurecedora para Lastic® Xtra Putty, Lastic® function Medium y Lastic® 90 Fine.
Lastic® Xtra Putty, Lastic® function Medium ve Lastic® 90 Fine için katalizör patı.
Pasta catalisadora para Lastic® Xtra Putty, Lastic® function Medium e Lastic® 90 Fine.
Pasta utwardzająca do mas Lastic® Xtra Putty, Lastic® function Medium i Lastic® 90 Fine.
Lastic® Xtra Putty, Lastic® function Medium 및 Lastic® 90 Fine용 경화제.

Lastic® Xtra Putty, Lastic® function Medium, Lastic® 90 Fine用の硬化剤ペースト

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Lastic® Universal hardener paste acts as an activator in combination with the particular Lastic® base impression material (Lastic® Xtra Putty, Lastic® function Medium, Lastic® 90 Fine).

Warnings

Lastic® impression materials should not be used with addition-curing silicones, Vinylsiloxanether®, polyether, or polysulfide materials.

Lastic® impression materials must always be combined with Lastic® Universal hardener paste.

Cautions

Do not use after expiration date.

Do not swallow. If the material is inadvertently swallowed: Please seek medical advice if symptoms develop or persist.

Avoid contact with the eyes. If the material inadvertently makes contact with the eyes: Flush immediately with an eye bath or plenty of water. Please seek medical advice if symptoms develop or persist.

These products do not normally cause allergic reactions; however, in sensitive persons an allergic reaction to this product cannot be ruled out.

The material must not enter the sewerage system or the water supply system to prevent damage to the environment.

Avoid contact with clothing, as the material cannot be removed by dry cleaning.

To obtain optimal impressions, the product temperature before use must not deviate significantly from 23 °C (74 °F). The processing and intraoral setting times may otherwise be affected.

Product name	Dosage with Lastic® Universal hardener paste	
	Product	hardener paste
Lastic Xtra® Putty	1 dispensing scoop (12.3 mL)	13 cm strand length (0.5 mL)
Lastic® function Medium	10 cm strand length (7.9 mL)	10 cm strand length (0.4 mL)
Lastic® 90 Fine	10 cm strand length (3.9 mL)	10 cm strand length (0.4 mL)

Instructions

A dose of Lastic® Universal hardener paste that deviates by up to 25% more or less shortens or prolongs respectively not only the setting time but also the processing time. Do not exceed the dose of Lastic® Universal hardener paste by more than 25% under any circumstances.

Storage conditions

Store in a dry place, protected from sunlight and at room temperature, avoiding extreme temperatures.

Disinfection

Impressions can be disinfected with a 2% glutaraldehyde solution or with disinfection solutions specifically for impression materials (e.g., Silosept®). Follow the manufacturer’s instructions.

The impressions can undergo galvanic silver plating on an alkaline solution.

Pouring the impressions

The impression can be poured within 2 to 24 hours using standard dental stone (classes III–V) to prepare models.

Trademarks

Lastic® and Silosept® are registered trademarks of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials/products are available in certain markets only.

For professional use only. Caution: Federal (USA.) Law restricts this device to sale by or on the order of a dentist.

Limitation of liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the legal basis asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology.

Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use of the product.

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