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## English

**Product description / Indication**

Visalys® Tooth Primer is a self-etching single-component primer with acidic polymers (10-MDP) for adhesive bonding of Visalys® CemCore and Visalys® Core to the dental hard substance. The primer can be applied without additional etching or after selective etching the enamel. Do not light cure Visalys® Tooth Primer. BPA or BPA precursors are not used in the manufacturing process of this device.

**Indications for use**

For permanent cementation of tooth structure to:

- crowns, bridges, inlays, onlays, veneers and adhesive bridges in combination with Visalys® CemCore
- root posts in combination with Visalys® CemCore and Visalys® Core
- indirect core build-ups in combination with Visalys® CemCore

For build-up adhesive bond to structure with Visalys® CemCore and Visalys® Core to fabricate:

- adhesive core build-ups

**Contraindications**

In the case of known allergies to methacrylate monomers, especially HEMA or other constituents of Visalys® Tooth Primer. If there are no adequate means of ensuring drainage.

**Warnings**

Allergic reactions to the product cannot be ruled out in sensitive persons. Sensitization through contact with skin / mucous tissue is possible.

The use of hydrogen peroxide solutions, materials containing eugenol or retraction paste containing iron(III) sulfate can impair adhesion.

In the case of vital teeth, areas near the pulp may have to be treated with suitable pulp / dentin protection.

Keep out of reach for children.

**Cautions**

Do not use after the expiration date.

Do not swallow material. If the material is inadvertently swallowed: Consult a medical doctor if problems arise or persist.

Take the appropriate protective measures (e.g. wear gloves or safety glasses). Avoid contact of skin and mucous tissue with the material.

Avoid contact with the eyes. If there is accidental contact with the eyes: rinse immediately and thoroughly with eye wash or water. Consult a medical doctor if problems arise or persist. Avoid contact with clothing, as permanent damage to clothes cannot be excluded.

Safety data sheets are available at [www.kettenbach.com](http://www.kettenbach.com).

**General notes**

Given refrigerator storage, the product has to have reached room temperature prior to use.

Visalys® Tooth Primer is only intended for pretreatment of natural dental hard substance primer and not for surface treatment of prosthetic restorations, indirect core build-ups and implant abutments.

Only use Visalys® Tooth Primer in combination with Visalys® CemCore and Visalys® Core.

**Use of dental hard substance primer to cement with Visalys® CemCore**

Visalys® Tooth Primer is a self-etching primer and can therefore be used without additional phosphoric acid etching. Only with cementation of veneers and adhesive bridges and with uncut enamel is selective phosphoric acid etching of the enamel necessary. Depending on the clinical conditions, in other cases too selective acid etching may be optionally performed. Use the etching gel according to the instructions for the relevant manufacturer.

- Following the try-in of the restoration, saliva, blood and any residues of the Visalys® CemCore Try In Paste must be removed from the adhesive surfaces of the dental hard substance using water spray and then keep the surfaces dry. The adhesive surfaces must be free of contamination. A rubber dam is recommended for a dry operating field.
- Drop Visalys® Tooth Primer into a preparation tray. Light protection is not necessary. Close primer bottles carefully after each use.

- Apply Visalys® Tooth Primer to the entire adhesive surface using a suitable applicator and rub in for 20 sec. After applying the Tooth Primer, start with the further processing immediately. Any excess can be removed with a cotton pellet and blown off with air. Then dry with a gentle, oil-free jet of air. The prepared primer should be processed within 5 min., as constituents can evaporate.

Visalys® CemCore must then be applied according to the instructions for use of the product.

**Note**

For cementation on an implant abutment or on an indirect core build-up, the specifications of the respective material manufacturer must always be considered. Pretreat the entire adhesive surfaces of the abutment / core build-up dependent on the material, as described under “Pretreatment of the restoration” in the Visalys® CemCore instructions for use, unless prescribed otherwise in the relevant material manufacturer’s instructions for use.

The emergence profiles and screw channels, as applicable, must be protected by suitable measures (e.g. wax, silicone covers).

**Cementation of root posts (Visalys® CemCore and Visalys® Core) and indirect core build-ups (Visalys® CemCore)**

- Prepare the root canal, root post or indirect core build-up according to the instructions for use from the relevant manufacturer.
- Wetten the cleaned and dried surfaces of the root canal, the coronal dentin and the enamel parts, as applicable, completely with Visalys® Tooth Primer and rub in for 20 sec. Remove excess from the root canal with a paper tip and dry all surfaces with a gentle jet of air.
- Apply Visalys® CemCore or Visalys® Core according to the instructions for use of the product and cement the root post or indirect core build-up (only Visalys® CemCore).
- Perform the build-up of the core as described under “Instructions for use for core build-up” from the Visalys® CemCore or Visalys® Core instructions for use.

**Application instructions as dental hard substance primer for core build-up**

- Clean and dry the cavity.
- Areas close to the pulp must be protected with the appropriate means.
- Ensure a clean and dry operating field (rubber dam) during the entire application.
- Selective phosphoric acid etching of the enamel may be undertaken optionally.
- Drop Visalys® Tooth Primer into a preparation tray. Light protection of the primer is not necessary.
- Apply Visalys® Tooth Primer to the entire adhesive surface of the core or the cavity using a suitable applicator and rub in for 20 sec. Any excess can be removed with a cotton pellet and blown off with air. Then dry with a jet of air. The prepared primer should be processed within 5 min., as constituents can evaporate.
- Perform the build-up of the core as described under “Instructions for use for core build-up” from the Visalys® CemCore instructions for use or the Visalys® Core instructions for use.

**Cleaning and disinfection**

The Visalys® Tooth Primer bottle can be cleaned with wipe disinfection using non-oxidizing disinfectants. Disinfection of the bottle in accordance with national guidelines. Observe standard practice hygiene measures.

**Storage conditions**

Store dry, protected from sunlight. The product has to be kept in the refrigerator at 2-8 °C/36-46°F before first use.

After the first use, Visalys® Tooth Primer has to be used within 6 months. Avoid extreme temperatures. Given refrigerator storage, the product has to have reached room temperature prior to use.

Do not use after the expiration date.

**Constituents**

The main components of the Visalys® Tooth Primer are water, acidic monomer (10-MDP) and HEMA.

**Disposal**

Dispose of the product in accordance with local regulations.

**Trademarks**

Visalys® Tooth Primer, Visalys® CemCore Try In Paste, Visalys® CemCore and Visalys® Core are registered trademarks of Kettenbach GmbH & Co. KG.

**Note**

Not all Kettenbach products are available in all countries.

For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

**Limitation of liability**

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the legal basis asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use of the product.

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